**FI**

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ASK-60

ASK-70

ASK-85

EN

Installation
Instructions

Supporting
fold-out rail

ASK-60

ASK-70

ASK-85

SE

Installationsanvis-
ningar

Utfällbart
stödhandtag

ASK-60

ASK-70

ASK-85

FI

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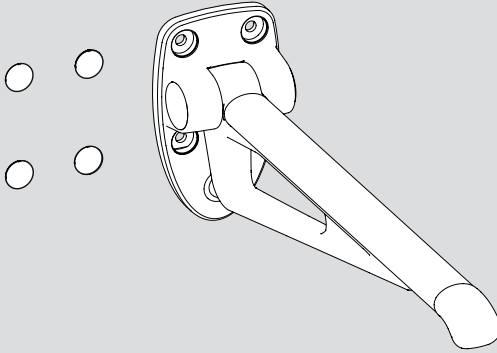
EN

Scope of Delivery

SE

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1

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Single components:

1 Supporting fold-out rail
4 Cover caps

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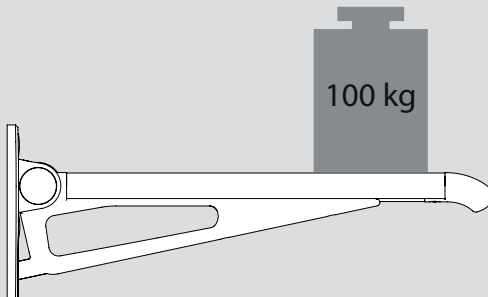
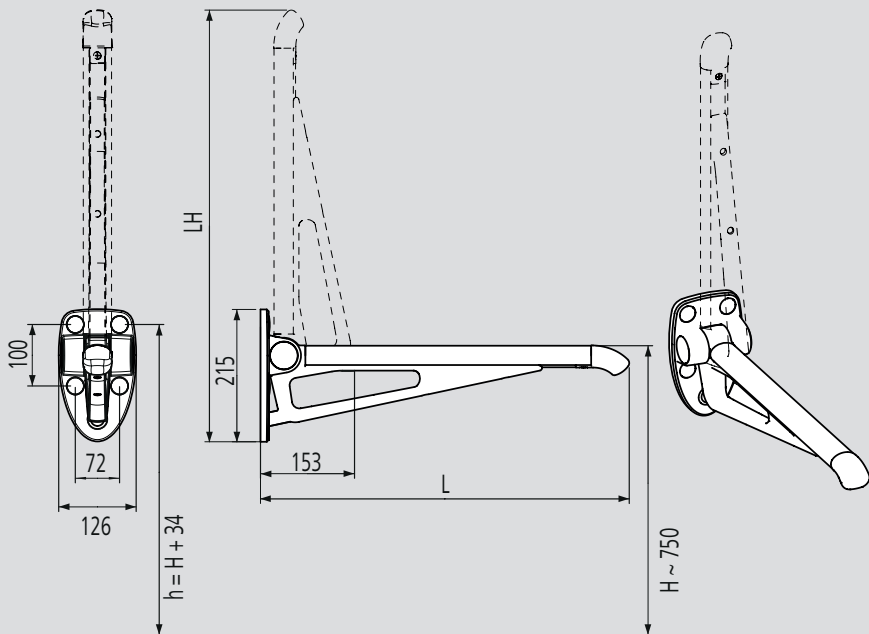
FI

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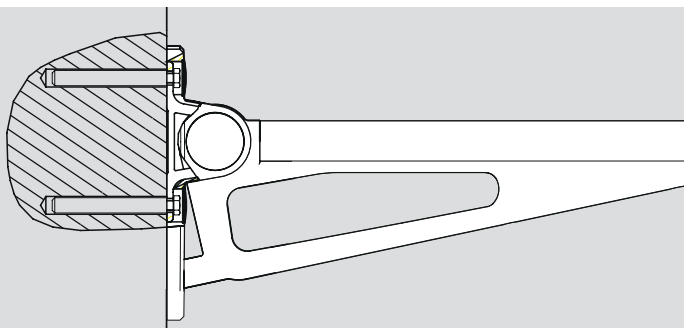
EN

Mounting, Installation Instructions

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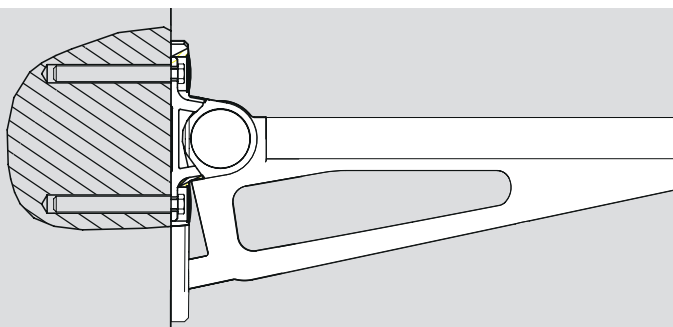
Recommended mounting systems for concrete, free of cracks \geq C20/25: Fischer FIS EM; Hilti HIT-HY 150; MKT VMU TOX TVM-STV; Spit EPOMAX; Mungo MIT-SE plus. Use anchors with interior thread, preferably made of stainless steel. Use screws, ISO4017 (DIN933) M8 A2-70 or M10 A2-70. The wall must have sufficient load-bearing capacity. Wall mounting not included in the scope of delivery. Installation by qualified personnel only.

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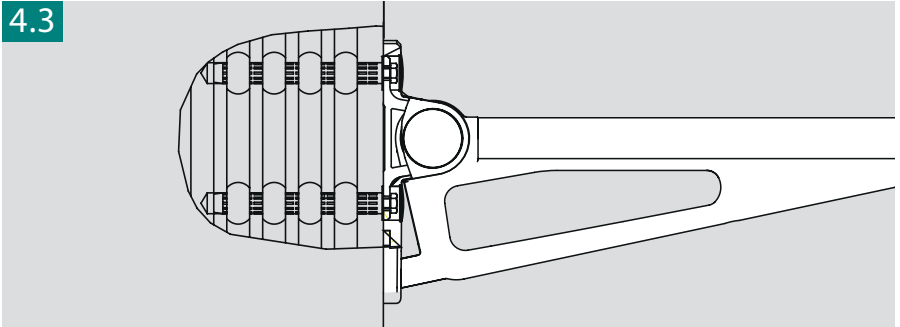
Recommended mounting systems for solid brick Mz12, Ks12: Fischer FIS V; Hilti HIT-HY 70; MKT VMU; TOX TVM-STV; Spit EPOMAX; Mungo MIT-SE plus. Use anchors with interior thread, preferably made of stainless steel. Use screws, ISO4017 (DIN933) M8 A2-70 or M10 A2-70. The wall must have sufficient load-bearing capacity. Wall mounting not included in the scope of delivery. Installation by qualified personnel only.

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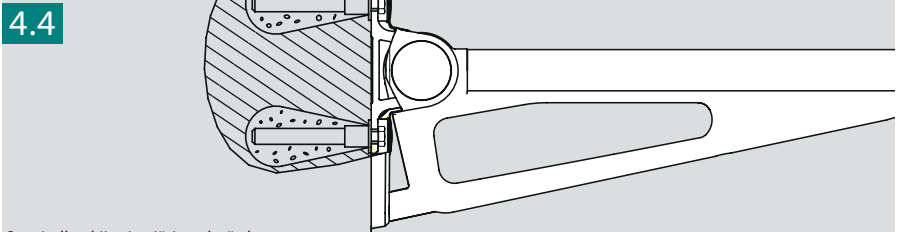
Recommended mounting systems for perforated brick HLz12, KSL12, Hbn4, Hbl4: Fischer FIS V; Hilti HIT-HY 70; MKT VMU; TOX TVM-STV; Spit EPOMAX +; Mungo MIT-SE plus. Use anchors with interior thread, preferably made of stainless steel. Use perforated sleeves. Use screws, ISO4017 (DIN933) M8 A2-70 or M10 A2-70. Only drill in rotary mode. The wall must have sufficient load-bearing capacity. Wall mounting not included in the scope of delivery. Installation by qualified personnel only.

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Recommended mounting systems for porous concrete; Fischer FIS V. Use anchors with interior thread, preferably made of stainless steel. Use centring sleeve. Use tapered bit. Use screws, ISO4017 (DIN933) M8 A2-70 or M10 A2-70. The wall must have sufficient load-bearing capacity. Wall mounting not included in the scope of delivery. Installation by qualified personnel only.

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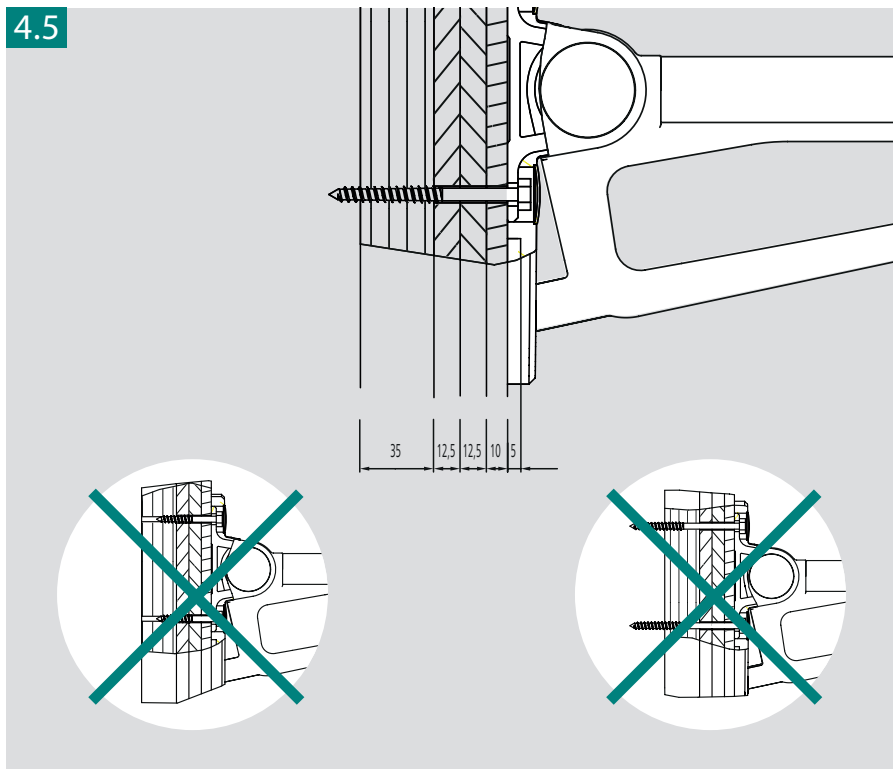
Recommended mounting systems for prefabricated construction: use front-wall installation systems from renowned manufacturers. For mounting on wood-based panels, use hexagon head wood screws \varnothing 8 A2-70. For mounting on front-wall installation systems with prefabricated metric thread, use screws ISO4017 (DIN933) M8 A2-70. The screw length depends on the front-wall system. The specifications of the front-wall installation system manufacturer and the screw manufacturer must be observed for mounting. Installation by qualified personnel only.

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EN

Cleaning and Care Instructions

To guarantee the long life and ongoing visual attractiveness of your Erlau product, we recommend gently cleaning the components regularly.

Please observe the following:

- ▲ Only use cleaning agents intended for cleaning bathroom fixtures.
- ▲ Avoid highly acidic substances, e.g. those containing hydrochloric acid, sulphuric acid, phosphoric acid, acetic acid or formic acid.
- ▲ Avoid highly basic substance, e.g. Natriumhydroxid, bleach, industrial cleaners, floor cleaners and grill or pipework cleaners as well as solvents such as acetone.
- ▲ Use a soft cloth for cleaning, e.g. microfibre, and avoid scouring agents, scouring brushes, scouring or kitchen sponges since these can cause surface damage.
- ▲ Do not spray the cleaning agent directly onto the product; apply it with a cleaning cloth.
- ▲ After cleaning, rinse the product with clear water or wipe off remaining cleaning agent with a cloth.
- ▲ Remove dirt and residues of personal hygiene and cleaning agents such as soap and shampoo from the product as soon as possible, so these do not stick to the surface.
- ▲ Observe the directions of the cleaning agent manufacturer (concentration, application time).
- ▲ Surfaces that have been damaged during use may suffer additional damage from the effects of cleaning agents of any kind.

Improper handling or incorrect care can cause surface pitting and damage. Any resulting damage is not covered under the warranty.

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EN

Liability

§ The manufacturer is not liable for damages and / or consequential damages due to improper use, long-term storage, improper installation or improper care.

In any case, the manufacturer's liability is limited to gross negligence and intent.

The applicable legal provisions apply in all other cases.

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The manufacturer offers a warranty on the product pursuant to the following principles and legal regulations.

The warranty can only be honoured with the presentation of a properly completed warranty certificate!

The warranty, which is granted by the manufacturer pursuant to the scope specified in the respective current version of the German Civil Code (BGB), entitles the buyer to have the purchased product repaired free of charge. Rights arising from product defects covered by the warranty term expire if they are not asserted during the warranty term. The warranty term is extended by the time it takes to process a complaint, insofar as the complaint is justified. The customer is obligated to assert the warranty exclusively against the seller specified on the warranty certificate. It is in your own best interest to fully

Recommendation

Keep the warranty certificate in a safe place along with the purchase receipt (or proof of proper product installation) for the entire warranty term. You must provide the original documents in case of a complaint.

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Warranty, Warranty Terms and Conditions

Warranty Exclusions

The warranty does not extend to:

- Damage due to improper installation
- Damage due to improper use or maintenance
- Excessive product wear
- Mechanical or other damage to the product in unexpected situations (e.g. fire)
- Damage due to the use of the product under unsuitable or extreme conditions (e.g. in the sauna, outdoors etc.)

Warning!

The supporting fold-out rails of Erlau AG are designed for a maximum static load of 100 kg (220 lbs.). Excess load may cause irreparable damage to the rail. In case of an overload, the rail has to be inspected by a specialist prior to further usage. The supporting fold-out rail is exclusively designed for the intended purpose in accordance with Directive 93/42/EEC. Alternative usage of the rail, e.g. as play and climbing device, may cause severe accidents and injuries to the user and also completely destroy the supporting fold-out rail. During usage, no persons may stay within the pivoting range of the supporting fold-out rail as this may cause injuries, particularly if unsupervised children use it. Due to the characteristics of the material, the rails have a certain flexibility for support comfort and reduction of injury in case of bumping into them. If unexpected permanent distortion occurs, please have the rail checked by a specialist prior to further use. Please note that the components of the supporting fold-out rail may heat up upon contact with hot water and that the user may sustain burns during subsequent use.

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EN

Warranty, Warranty Terms and Conditions

Installation of the supporting fold-out rail

The safe and sound function of the supporting fold-out rail is only ensured by proper installation on the wall. For this reason, the installation must be performed by qualified experts (fitters). Improper installation may cause damage to the rail and lead to severe injuries to the user. We therefore advise against self-installation. During the installation, the experts should particularly ensure that the installation surface is level and has no elevations. The support structure of the wall should be suitable for the load capacity of the rail and should not have weak points. It is furthermore important to ensure usage of the correct mounting material. Please observe these mounting instructions as well as the instructions of the screw and plug manufacturers. A functional test must be performed immediately after installing the rail.

Purpose according to Directive 93/42/EEC

Supporting fold-out rails are required as support during standing, sitting, getting up and specifically while moving from wheelchair to toilet or shower stool. It is used mainly by persons no longer able to perform daily personal hygiene tasks without supporting aids due to their physical limitations. According to need, supporting fold-out rails are installed in the toilet area, toilet seat, next to the basin or in the shower.

Warranty period

The warranty period commences on the date on which the purchaser acquired the product. The date is indicated as the date of the sale in the warranty certificate or as the date on which the service technician installed the product (if the installation was performed by a service technician within three weeks following purchase). The warranty period is 24 months.

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Vakiometalli Oy